MED LON

### **Lumbastab Rugbraces**

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EN – English

**Dear Customer,**

Thank you for choosing a Medilon Lumbastab Back Brace. This medical device is designed to support and stabilize the lower back for various medical indications.

Please read these instructions for use carefully before first use and keep this document for future reference.

| **Note:** The brace may feel tight during the initial fit; this is normal for supportive compression. Use the hook-and-loop fasteners to adjust the correct pressure and fit.

### **Intended Use**

The Medilon Lumbastab Back Brace is a non-invasive medical device designed to support and stabilize the lumbar spine. The brace provides compression and structural support through semi-rigid stays and adjustable straps. This may help reduce load on the lower back, contributing to pain relief and recovery.

The brace can be worn during daily activities, work, or sports that place strain on the lower back.

### **Indications:**

- Non-specific lower back pain
- Overuse of the lower back
- Herniated disc (disc-related conditions)
- Osteoarthritis of the spine
- Rheumatic conditions
- Lumbar instability
- Rehabilitation after back injury or surgery
- Support during daily activities (sitting, lifting, bending)
- Preventive support during physically demanding work or sports

### **Application Area**

The back brace is intended for use as described in the indications. Use the brace only on the lower back (lumbar spine) and according to the guidelines in this manual.

The brace is worn around the lower back, at the level of the natural curve (lordosis) of the spine.

### **Risks and Warnings**

#### **⚠ General Instructions**

Follow the instructions in this manual carefully. When used correctly, there are no known risks, provided that:

- The product is used only according to the indications
- The device is applied correctly
- The correct size is selected

#### **⚠ Skin Irritation**

Prolonged use may cause redness or itching. The breathable material helps to reduce this risk.

#### **⚠ Compression**

- Overtightening may cause pressure discomfort or irritation
- Ensure the brace fits firmly but remains comfortable and does not restrict breathing
- Check the skin for redness or irritation during prolonged use
- Loosen the brace if it becomes uncomfortable

#### **⚠ Product Modifications**

Do not modify the product, as this may reduce its effectiveness or be harmful to health; in such cases, warranty and liability are void.

#### **⚠ Contact with Substances**

Avoid contact with fatty or acidic substances, ointments, or lotions, as these may affect the material properties of the product.

#### **⚠ Changes in Symptoms**

If you notice changes or worsening of symptoms while wearing the product, stop use immediately and consult your doctor or healthcare professional.

#### **⚠ Incorrect Use or Damage**

Incorrect use or damaged materials may reduce or eliminate the intended therapeutic effect.

### **Contraindications**

Do not use this product without consulting a doctor or qualified healthcare professional in the following cases:

- Severe skin conditions, including open wounds, skin infections, eczema, or inflammation in the area of use
- Known allergies to the materials used
- Circulatory disorders, such as severe peripheral vascular disease
- Neurological conditions associated with reduced sensation, tingling, or numbness

(including diabetic neuropathy)

- Severe or unexplained swelling in the lower back or abdominal region
- After a recent fracture, surgery, or serious injury, unless advised otherwise by a doctor

### **⚠ Warning**

Stop use immediately and consult a doctor if symptoms occur during or after use, such as increased pain, numbness, tingling, skin discoloration, swelling, or other adverse reactions.

### **Considerations and Medical Advice**

In cases of underlying medical conditions, the use of this device should be carefully considered. Consult a doctor or qualified healthcare professional to determine whether this product is suitable for your situation and which type of support is most appropriate. This product is intended for support and does not replace medical advice or treatment.

Failure to follow the instructions and warnings in this manual may result in reduced effectiveness or unwanted effects.

### **Instructions for Use**

#### **Adjusting the Product**

This product is designed for easy independent use. Ensure that the back brace is applied firmly yet comfortably and provides adequate support without restricting breathing. The brace should not be overtightened.

Consult a doctor or qualified healthcare professional if you are unsure about the correct size, fit, or suitability of the product.

#### **Applying the Back Brace**

Apply the back brace over a clean, dry undershirt or T-shirt. Do not use the brace directly on wet skin or on skin treated with creams, oils, or ointments, as this may affect the material and cause skin irritation. Read the full instructions carefully before use.

#### **Maintenance and Storage**

Store the back brace in a clean, dry place, protected from direct sunlight and heat sources. Check the product before each use for wear, damage, or reduced performance, such as worn fasteners or deformed support elements. Do not use the product if it no longer functions properly.

## **Product Lifespan**

Regular use may lead to natural wear and tear of the back brace. The lifespan depends on frequency of use, load, and maintenance. The expected lifespan is approximately 6–12 months.

The product should be replaced when support decreases or when visible damage occurs.

## **Washing Instructions**

- Hand wash or gentle cycle max. 30 °C
- Close hook-and-loop fasteners before washing
- Do not use bleach, fabric softeners, or optical brighteners
- Do not tumble dry or dry using direct heat
- Allow the brace to air dry
- Clean hook-and-loop fasteners and inner lining with a damp cloth if necessary

## **Reuse Instructions**

The back brace is intended for individual use by one person. The product may be reused by the same user if properly maintained and stored. Sharing with others is not permitted due to hygiene and health risks.

## **Warranty**

The legal provisions of the country in which the product was purchased apply.

In case of a warranty claim, please contact the seller.

- The product must be cleaned before submitting a warranty claim
- Warranty may be limited or void if usage or maintenance instructions are not followed
- Warranty does not apply in cases of misuse, failure to follow medical advice, or unauthorized modifications

## **Liability**

Use this product only as described in this manual. Consult a doctor or qualified healthcare professional if you have doubts about its use or in case of existing medical conditions.

This product is intended to support and relieve the back. It does not replace medical treatment. The manufacturer is not liable for damage or injury resulting from improper use, ignoring instructions, or unauthorized modifications.

## **Incident Reporting**

According to applicable regulations, any serious incident occurring in relation to the use

of this back brace must be reported immediately to the manufacturer and the competent authority. Our contact details can be found on the back of this manual.

## **Disposal**

At the end of its lifespan, the product may be disposed of in accordance with local regulations. The back brace may be disposed of with regular textile waste. For sustainable processing, it can be returned to a textile collection point where medical textiles are safely recycled.

## **Material Composition**

The back brace consists of the following materials and components:

- Polyester (shape-retaining and durable)
- Nylon (flexible and breathable)
- Stays (supportive reinforcement)
- Hook-and-loop fasteners

[MD] – Medical Device

[UDI] – Unique Device Identification (UDI-DI)

Date of information: June 2025

## **CE Marking and Standards**

- CE marking: Class I (MDR 2017/745)
- Applied standards: ISO 13485, ISO 14971, EN ISO 15223-1, EN ISO 20417

## **Traceability and PMS**

- Each product is provided with a LOT number / UDI-DI on the packaging
- Post-Market Surveillance (PMS) plan: complaint registration, returns, feedback analysis
- Annual analysis and risk assessment are conducted to ensure product safety

## **Tips for Application**

Correct application of the back brace is important for optimal support of the lumbar spine and wearing comfort. Ensure the brace fits firmly yet comfortably without restricting breathing.

1. If possible, wear a thin layer of clothing under the brace, such as an undershirt or T-shirt.
2. Loosen all hook-and-loop fasteners.

3. Position the brace around the lower back so that the center aligns with the natural curve (lordosis) of the spine.
4. Ensure the stays are centered along the spine. For models with multiple stays, these should be evenly distributed for optimal support.
5. Apply the brace while standing upright in a relaxed posture.
6. Close the front and tighten the elastic straps evenly to achieve firm but comfortable compression. Adjust compression based on comfort and freedom of movement; the brace should not be too tight.
7. Check that the brace fits properly and provides sufficient support during daily activities. Adjust if necessary.
8. If in doubt about size, positioning, or compression, consult a doctor or qualified healthcare professional.

Also follow the instructions on the inside of the first page of this brochure.

### **Tips for Removal**

1. Loosen the hook-and-loop fasteners.
2. Carefully remove the brace.
3. Check the skin for redness or irritation.
4. Close the fasteners before storing the brace.

### **Wearing Time**

- Wear during activities that strain the lower back, such as work or sports
- Remove the brace during rest or at night unless otherwise advised by a doctor or physiotherapist
- Reduce usage as symptoms improve

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