MED LON

Carpostab Pro Polsbrace

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EN – English

Dear Customer,

Thank you for choosing the Medilon Carpostab Pro Wrist Brace. This medical device is designed to stabilize and support the wrist for various medical indications.

Please read these instructions carefully before first use and keep this document for future reference.

| Note: The brace may feel tight at first; this is normal for semi-rigid support. Use the hook-and-loop straps and elastic band to adjust the correct compression.

Intended Use

The Medilon Carpostab Pro Wrist Brace is a semi-rigid, non-invasive medical device designed to stabilize, immobilize, and support the wrist for a variety of conditions. Its purpose is to keep the wrist in a neutral, relaxed position, preventing overuse, irritation, and compression of nerves or tendons.

The brace is suitable for both acute and chronic conditions and can be worn during the day or at night, depending on the nature of the complaints.

Indications:

- Overuse injuries (RSI)
- Tendon inflammations (tendinitis)
- Carpal tunnel syndrome (CTS)
- Osteoarthritis and rheumatism
- Instability complaints (including during pregnancy)
- TFCC injury, ganglion
- Sprains, strains, and contusions
- Post-traumatic or postoperative rehabilitation
- Preventive support for high-risk activities

Place of Use

The wrist brace is intended for use as described in the indications. Use the brace only on the designated body parts and according to the guidelines in this manual.

Risks and Warnings

⚠ General Instructions

Please follow the instructions in this user manual and any advice from qualified personnel carefully. When used correctly, there are no known risks, provided that:

- The product is used strictly according to the indications;
- The device has been correctly measured;
- You have received instructions on the proper application and wearing of the wrist brace.

⚠ Skin Irritation

Prolonged wear may cause redness or itching; breathable material helps reduce this.

⚠ Compression

- Over-tightening may cause constriction or discomfort; use the adjustable straps correctly.
- Regularly check the skin and circulation in the hand and fingers.

⚠ Modifications to the Product

Do not make any modifications to the product, as this may reduce support or be harmful; in such cases, warranty and liability are void.

⚠ Contact with Substances

Avoid contact with oils, creams, or lotions, as these may affect the material properties of the product.

⚠ Changes in Complaints

If you notice any change or worsening of symptoms while wearing the brace, stop using it immediately and consult your doctor or treating specialist.

⚠ Incorrect Use or Damage

Incorrect use, wrong size, or damaged materials may reduce the desired therapeutic effect or cause skin problems.

Contraindications

This product should not be used without prior consultation with a physician or qualified healthcare provider in the following cases:

- Severe skin problems, including open wounds, skin infections, eczema, or inflammation in the area where the brace is worn
- Known allergies to the materials used
- Circulatory disorders, such as severe peripheral vascular disease
- Neurological conditions associated with reduced sensation, tingling, or numbness (including diabetic neuropathy)
- Severe or increasing swelling of the hand or wrist
- Recent fracture, surgery, or serious injury unless otherwise advised by a doctor

 **Warning:**

Stop use immediately and consult a doctor if during or after use you experience increased pain, numbness, tingling, skin discoloration, swelling, or other unwanted reactions.

Assessment and Medical Advice

In situations involving underlying medical conditions, use of this device should be carefully considered. Consult a doctor or qualified healthcare provider to determine whether this product is suitable for your condition and which type of support is most appropriate.

Failure to follow the instructions and warnings in this manual may result in reduced effectiveness or undesired effects.

Instructions for Use

Adjusting the Product

This product is designed for easy self-use. Ensure the wrist brace is applied firmly but comfortably, without restricting blood circulation. Do not tighten the brace too much.

Consult a doctor or qualified healthcare provider if you are unsure about the correct size, fit, or suitability of the product.

Applying the Wrist Brace

Apply the brace to clean, dry skin. Do not use on wet or cream-, oil-, or ointment-covered skin, as this may damage the material and cause skin irritation. Read the full instructions carefully before first use.

Maintenance and Storage

Store the wrist brace in a clean, dry place, protected from direct sunlight and heat

sources. Check the product before each use for wear, damage, or reduced function, such as worn closures or deformed reinforcement elements. Do not use the product if it no longer functions correctly.

Product Lifespan

Regular use may cause natural wear of the wrist brace. The lifespan depends on frequency of use, load, and maintenance. Recommended lifespan is approximately 6–12 months. Replace the product when support or stability is reduced or visible damage occurs.

Washing Instructions

- Hand wash or delicate cycle max. 30 °C
- Close Velcro before washing
- Do not use bleach, fabric softeners, or optical brighteners
- Do not tumble dry or expose to direct heat
- Air dry only
- Clean hook-and-loop fasteners and interior if necessary with a damp cloth

Reuse Instructions

The wrist brace is intended for individual use by one person. It can be worn repeatedly by the same user if properly maintained and stored. Sharing with others is not allowed due to hygiene and health risks.

Warranty

Applicable laws of the country of purchase apply. For any warranty claims, contact the seller.

- The product must be cleaned before processing a warranty claim
- Warranty may be limited or voided if instructions for use and maintenance are not followed
- Warranty does not cover improper use, failure to follow medical instructions, or unauthorized modifications

Liability

Use this product only as described in this manual. Consult a doctor or qualified medical professional if unsure about use or if existing medical conditions are present.

The product is intended to support and stabilize the wrist. It does not replace medical treatment. The manufacturer is not liable for damage or injury resulting from misuse, ignoring instructions, or unauthorized modifications.

Mandatory Reporting

Under applicable law, you are required to report any serious incidents occurring during the use of this wrist brace to the manufacturer and competent authority immediately. Our contact details are on the back of this manual.

Disposal

After the product's useful life, dispose of it according to local regulations. It is allowed to dispose of the wrist brace with regular textile waste. For sustainable disposal, return it to a collection point of Stichting UPV Textiel, where used medical textiles are safely recycled.

Material Composition

The wrist brace is made from the following materials and components:

- Main material: breathable, lightly elastic neoprene
- Interior: soft textile finish for optimal comfort
- Ventral splint (palm side): one rigid stay for immobilization
- Dorsal splints (back of hand side): three narrower stays for extra support
- Closure: hook-and-loop straps and elastic band around the wrist for precise compression and positioning

This construction stabilizes the wrist in a neutral position while maintaining comfort and mobility where possible.

[MD] – Medical Device

[UDI] – Unique Device Identification (UDI-DI)

Information Date: June 2025

More information can be obtained from your doctor, medical specialist, or the specialized store where the product was purchased.

CE Marking and Standards

- CE marking: Class I (MDR 2017/745)
- Applied standards: ISO 13485, ISO 14971, EN ISO 15223-1, EN ISO 20417

Traceability and PMS

- Each product has a LOT number / UDI-DI on the packaging
- Post-Market Surveillance (PMS) plan: complaint registration, returns, feedback analysis
- Annual analysis and risk assessment are performed to ensure product safety

Tips for Putting On

Proper application of the wrist brace is important for optimal support and comfort. Ensure the brace is correctly positioned around the wrist and straps are secure but comfortable.

1. Open all hook-and-loop straps and lay the brace flat
2. Place your hand in the brace with your thumb in the thumb opening
3. Adjust compression using the straps; start at the wrist and work toward the forearm
4. Ensure the brace is comfortable and fingers can move freely
5. Optionally use a thin sleeve (e.g., tubigrip) under the brace for extra comfort

Follow the instructions on the inside of the first page of this brochure.

Tips for Removal

1. Open the hook-and-loop straps from the forearm toward the wrist
2. Remove your hand carefully from the brace
3. Check the skin for redness, irritation, or other abnormalities
4. Close the straps before storing the brace to protect the material and extend lifespan

Wearing Time

- Depending on the nature of the complaints and advice from your doctor or therapist
- The wrist brace can be worn during the day or night

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