MED LON

### **Duimbrace**

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**EN – English**

## **Dear customer,**

Thank you for choosing the Medilon Thumb Brace. This medical device is designed to stabilize and support the thumb joint for various medical indications.

Please read these instructions carefully before first use and keep this document for future reference.

| Note: The brace may feel tight at first fit; this is normal for semi-rigid support. Use the Velcro straps to adjust the correct compression.

## **Intended Use**

The Medilon Thumb Brace is a semi-rigid, non-invasive medical device designed to stabilize, support, and partially immobilize the thumb joint for various indications. Its purpose is to maintain the thumb joint (including the CMC1 joint) in a functional, unloading position, thereby reducing overuse, irritation, and tendon impingement.

The brace is suitable for both acute and chronic conditions and may be worn during the day or at night, depending on the nature of the complaints.

## **Indications:**

- De Quervain's disease
- RSI complaints
- Thumb osteoarthritis (CMC1 joint)
- Skier's thumb (mild instability)
- Tendon inflammation around the thumb
- Postoperative support
- Overuse or irritation of thumb tendons
- Instability of the thumb joint
- Minor injuries (sprain, bruise)

## **Place of Use**

The thumb brace is intended for use as described in the indications. Use the brace only on the designated body parts and according to the guidelines in this manual.

## **Risks and Warnings**

### **⚠ General instructions**

Please follow the instructions in this user manual and the guidance of qualified personnel carefully. When used correctly, no known risks are associated, provided that:

- The product is used only according to the indications;
- The device is applied correctly;
- You have received instructions on the proper way to apply and wear the thumb brace;

### **⚠ Skin irritation**

Prolonged use may cause redness or itching; breathable mesh material helps limit this.

### **⚠ Compression**

- Excessive tightening can cause constriction or discomfort; use adjustable Velcro straps correctly.
- Regularly check the skin and circulation of the thumb and hand.

### **⚠ Modifications to the product**

Do not make modifications to the product, as this may reduce the expected support or be harmful to health; in such cases, warranty and liability are void.

### **⚠ Contact with substances**

Avoid contact with oily or acidic substances, creams, or lotions, as these can affect the material properties of the product.

### **⚠ Changes in complaints**

If you notice changes or worsening of symptoms while using the product, stop using it immediately and contact your doctor or healthcare professional.

### **⚠ Incorrect use or damage**

Incorrect use or damaged material may reduce or prevent the desired therapeutic effect.

## **Contraindications**

This product must not be used without prior consultation with a physician or qualified healthcare professional in the following cases:

- Severe skin problems, including open wounds, skin infections, eczema, or inflammation in the area where the brace is worn
- Known allergies to the materials used

- Circulatory disorders, such as severe peripheral vascular disease
- Neurological disorders with reduced sensation, tingling, or numbness (including diabetic neuropathy)
- Severe or increasing swelling of the thumb or wrist
- Recent fracture, surgery, or serious injury, unless advised otherwise by a physician

**⚠ Warning:**

Stop use immediately and consult a physician if during or after use you experience increased pain, numbness, tingling, discoloration, swelling, or other unwanted reactions.

## **Medical Advice and Considerations**

For underlying medical conditions, use of this device must be carefully considered. Consult a physician or qualified healthcare professional to determine if this product is suitable for your situation and which type of support is recommended.

Failure to follow the instructions and warnings in this manual may result in reduced effectiveness or unwanted effects.

## **Instructions for Use**

### **Adjusting the Product**

This product is designed for easy self-use. Ensure the thumb brace is applied firmly but comfortably and does not impair blood circulation. The brace must not be applied too tightly.

Consult a physician or qualified healthcare professional if you are unsure about proper fit or suitability of the product.

### **Applying the Thumb Brace**

Apply the brace to clean, dry skin. Do not use on wet skin or skin with cream, oil, or ointment, as this may affect the material and cause irritation. Read the full instructions carefully before use.

### **Maintenance and Storage**

Store the thumb brace in a clean, dry place, protected from direct sunlight and heat sources. Inspect the product before each use for wear, damage, or reduced function, such as worn closures or deformed splints. Do not use the product if it no longer functions correctly.

**Product Lifespan**

Regular use may lead to natural wear of the thumb brace. Lifespan depends on frequency of use, load, and maintenance. Recommended lifespan: approximately 6–12 months.

Replace the product when support or stability is reduced or visible damage occurs.

**Washing Instructions**

- Hand wash or gentle machine wash up to 30 °C
- Close Velcro before washing
- Do not use bleach, fabric softener, or optical brighteners
- Do not tumble dry or expose to direct heat
- Air dry flat
- Clean Velcro straps and interior with a damp cloth if necessary

**Reuse Instructions**

The thumb brace is intended for individual use by one person. It can be worn multiple times by the same user if maintained properly. Sharing with others is not allowed due to hygiene and health risks.

**Warranty**

Applicable law of the country of purchase applies. Contact the seller in case of a warranty claim.

- Product must be cleaned before warranty treatment.
- Warranty may be limited or void if usage and maintenance instructions are not followed.
- Warranty does not cover improper use, failure to follow medical personnel instructions, or unauthorized modifications.

**Liability**

Use this product only as described in this manual. Consult a physician or qualified medical professional if in doubt about use or existing medical conditions.

The product is intended to support and stabilize the thumb joint. It does not replace medical treatment. The manufacturer is not liable for damage or injury resulting from misuse, ignoring instructions, or unauthorized modifications.

## **Reporting Obligation**

You are required by law to report any serious incident occurring with the use of this thumb brace immediately to the manufacturer and competent authority. Contact details are on the back of this manual.

## **Disposal**

After the product's lifespan, dispose of it according to local regulations. It may be disposed of with regular textile waste. For sustainable processing, return to a collection point of Stichting UPV Textiel, where used medical textiles are safely recycled.

## **Material Composition**

The Thumb Brace is made of the following materials and components:

- Main material: breathable mesh material
- Interior: soft textile lining for optimal comfort
- Splints: two semi-rigid reinforcements on both sides of the thumb
- Closure: three adjustable Velcro straps for precise compression and positioning

This construction stabilizes the thumb joint in a functional position, maintaining comfort and mobility of the other fingers.

[MD] – Medical Device

[UDI] – Unique identification code of the DataMatrix (UDI-DI)

Information Date: June 2025

Further information is available from your physician, medical specialist, or the specialized store where the product was purchased.

## **CE Marking and Standards**

- CE Mark: Class I (MDR 2017/745)
- Applied standards: ISO 13485, ISO 14971, EN ISO 15223-1, EN ISO 20417

## **Traceability and PMS**

- Each product is marked with a LOT number / UDI-DI on the packaging.
- Post-Market Surveillance (PMS) plan: complaint registration, returns, feedback analysis.
- Annual analysis and risk assessment are performed to ensure product safety.

## **Tips for Putting On**

Proper application of the thumb brace is important for optimal support and comfort. Ensure the brace is positioned correctly around the thumb and hand, and straps are fastened firmly but comfortably.

Check after applying that the brace does not impair circulation.

1. Open all Velcro straps and lay the brace flat.
2. Insert the thumb through the designated opening.
3. Place the brace around the palm and back of the hand.
4. Fasten the Velcro straps and adjust compression until the brace is firm but comfortable.
5. Check that the thumb is supported and the other fingers move freely.  
Follow also the instructions on the inside of the first page of this brochure.

## **Tips for Removal**

1. Open the Velcro straps.
2. Carefully remove the brace from the thumb and hand.
3. Check the skin for redness, irritation, or other abnormalities.
4. Close the Velcro straps before storing the brace to protect the material.

## **Wearing Time**

- Depending on the nature of complaints and advice of your physician or therapist.
- The thumb brace can be worn during the day or night.

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